

Cynergy[™] Vascular Workstation

- Vascular lesions
- Pigmented lesions
- Facial and leg veins
- Rosacea
- Port wine stains
- Sun-damaged skin
- Wrinkle reduction



Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post 1 Tx, Spot size: 7mm, PDL 20 msec, 7 J/cm²
Delay Short, Nd:YAG 20 msec, 50 J/cm²

Photos courtesy of M. Goldman, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



PDL 10 msec, 7 J/cm², Delay Short,
Nd:YAG 10 msec, 40 J/cm²

Photos courtesy of P. Boixeda, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



PDL 10 msec, 7 J/cm², Delay Short,
Nd:YAG 10 msec, 40 J/cm²

Photos courtesy of P. Boixeda, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]




PDL 10 msec, 8 J/cm², Delay Short,
Nd:YAG 10 msec, 40 J/cm²

Photos courtesy of E. Tanghetti, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]

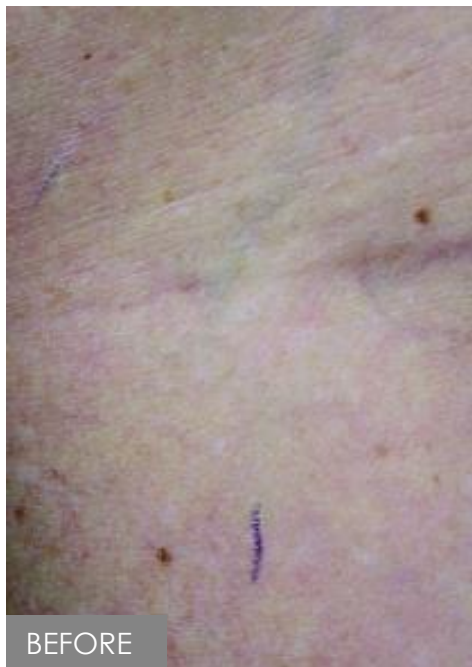

BEFORE



AFTER

Post 2 Tx, PDL 40 msec, 11 J/cm²,
Delay Medium, Nd:YAG 15 msec,
60 J/cm² (Blue reticular vein.)

Photos courtesy of R. Adrian, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post 1 Tx.

Photos courtesy of E. Tanghetti, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post 2 Tx, PDL 40 msec, 11 J/cm², Delay
Medium, Nd:YAG 10 msec, 65 J/cm².

Photos courtesy of R. Adrian, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post 2 Tx, PDL 20 msec, 8 J/cm², Delay Medium,
Nd:YAG 10 msec, 70 J/cm².

Photos courtesy of R. Adrian, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post 3 Tx, PDL 2 msec, 8 J/cm², Delay Medium,
Nd:YAG 10 msec, 30 J/cm².

Photos courtesy of M. Taylor, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[®]
Cynergy[™]

MultiPlex[™]





Post 1 Tx, PDL 0.5 msec, 7 J/cm², Delay Long, Nd:YAG
10 msec, 65 J/cm².

Photos courtesy of R. Geronemus, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]

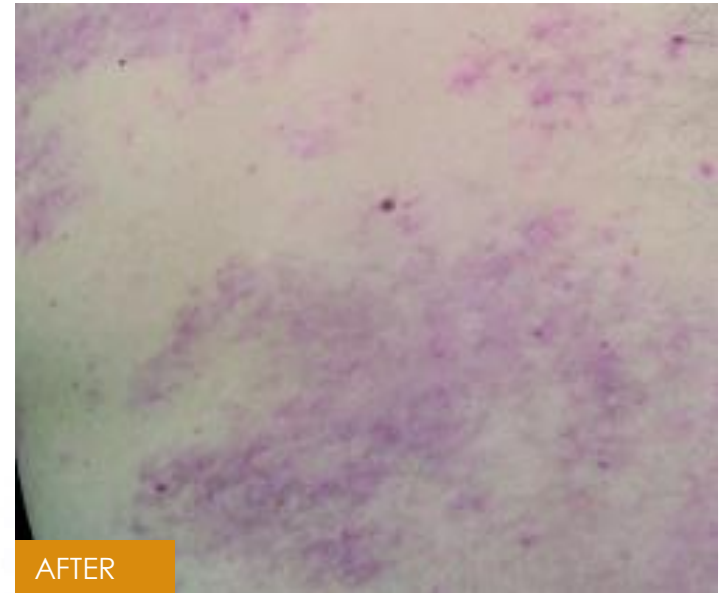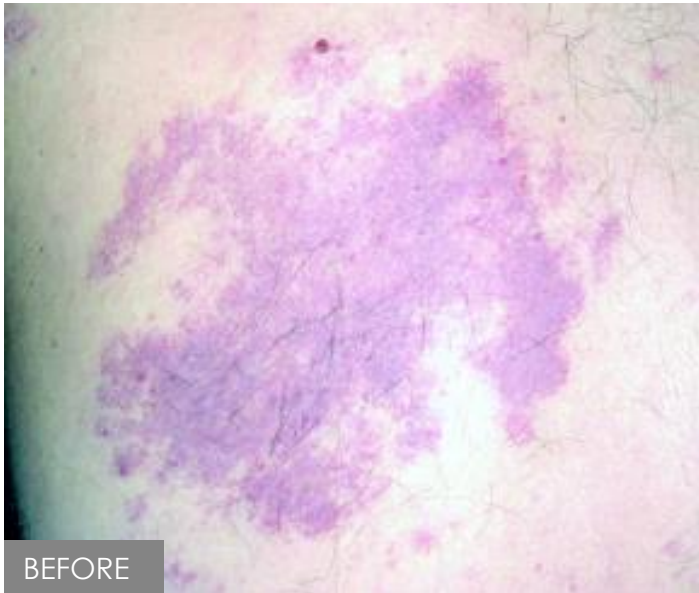

Post 3 Tx, PDL 10 msec, 8.5 J/cm², Delay Medium,
Nd:YAG 40 msec, 35 J/cm².

Photos courtesy of R. Geronemus, MD.

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]



Post, 2 Tx, PDL 20 msec 8 J/cm², Delay
Medium, Nd:YAG 15 msec, 55 J/cm²



Post, 1 Tx, PDL 20 msec 9 J/cm²,
Nd:YAG 15 msec, 45 J/cm²

Photos courtesy of Darcy Gilliard Marine, MD, Dermatologist, Orleans, France

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[™]
Cynergy[™]

MultiPlex[™]




Post 1 Tx, PDL 20 msec, 7.5 J/cm²,
Delay Long, Nd:YAG 20 msec, 35 J/cm².




Post 2 Tx.

Photos courtesy of Darcy Gilliard Marine, MD, Dermatologist, Orleans, France

Cynergy MultiPlex vs. Candela VBeam

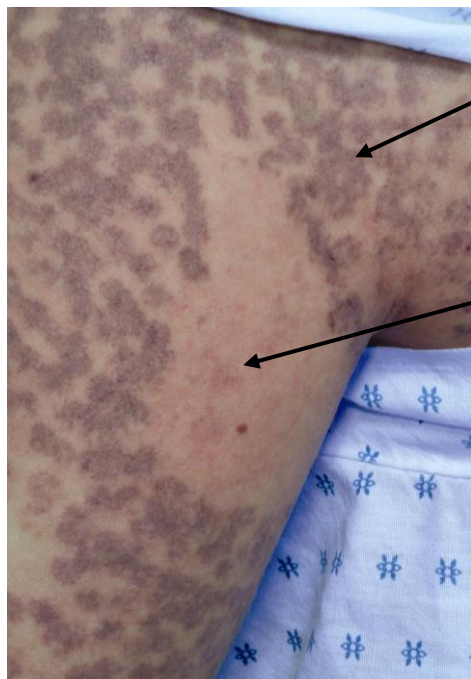

Candela V
Beam PDL 0.5ms
6.5 J/cm²
DCD 20/30

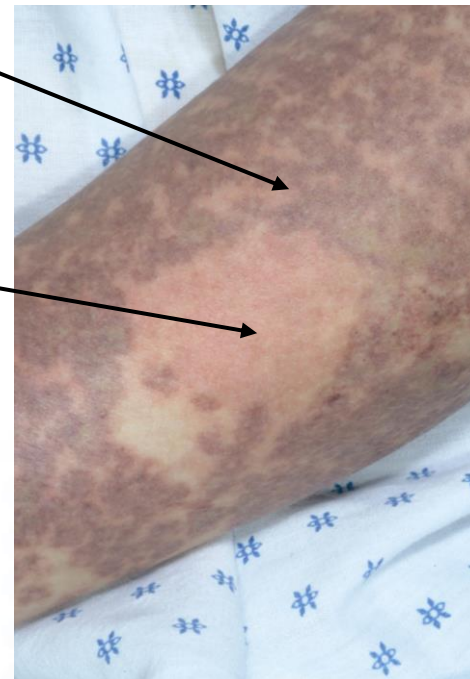

Cynosure
Cynergy with
MultiPlex
7mm spot
PDL .5 ms
7 J/cm²
Delay Medium
Nd:YAG 10 ms
40 J/cm²

This photo was taken post Tx.

This photo was taken post Tx.

Pulsed Dye Laser




Post 2 Tx, 40 msec, 12 J/cm²

Pulsed Dye Laser



Post 1 Tx, 40 msec, 14 J/cm²

Pulsed Dye Laser

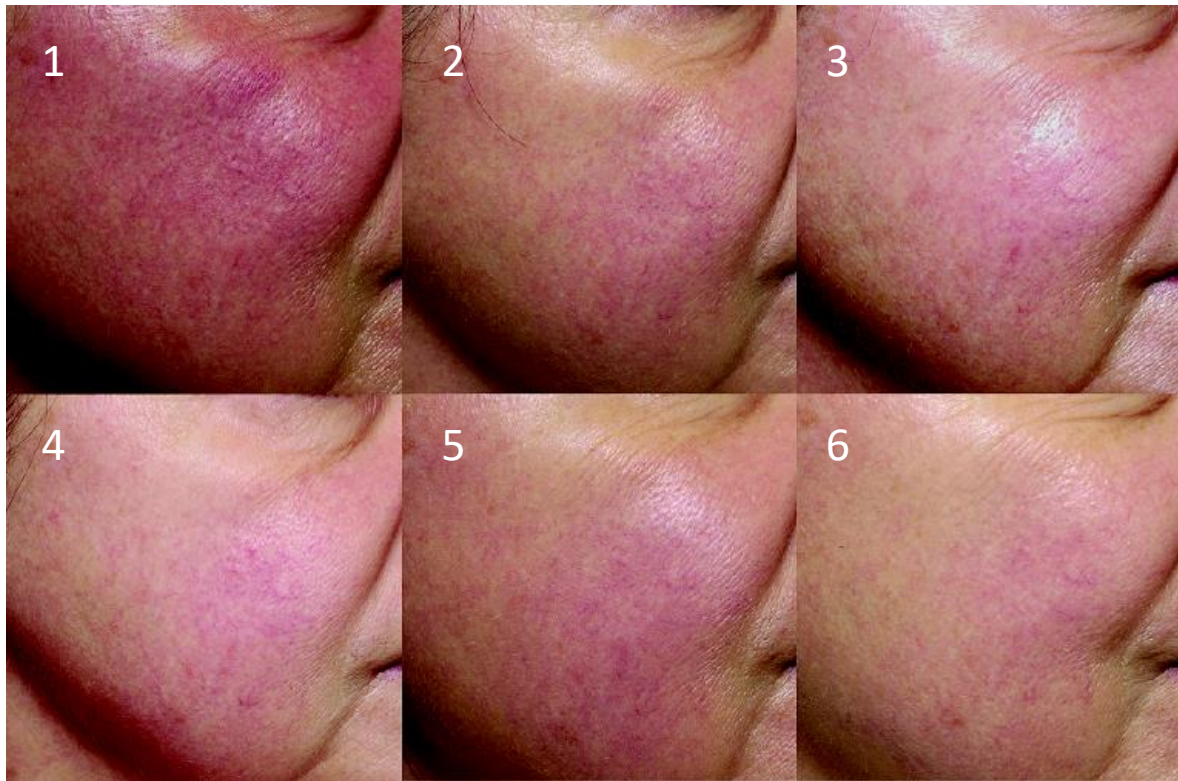

1. Before Tx.
2. Post 1 Tx.
3. Post 2 Tx.
4. Post 4 Tx.
5. Post 5 Tx.
6. One month post last Tx.



Treatment PDL 585 nm, 10 msec, 6 J/cm² (subpurpuric)

Photos courtesy of Dr. J. Lloyd.

Pulsed Dye Laser




Pulsed Dye Laser

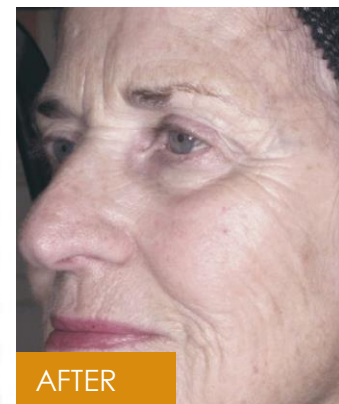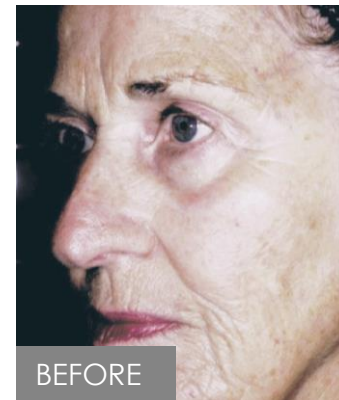


Pulsed Dye Laser




Pulsed Dye Laser

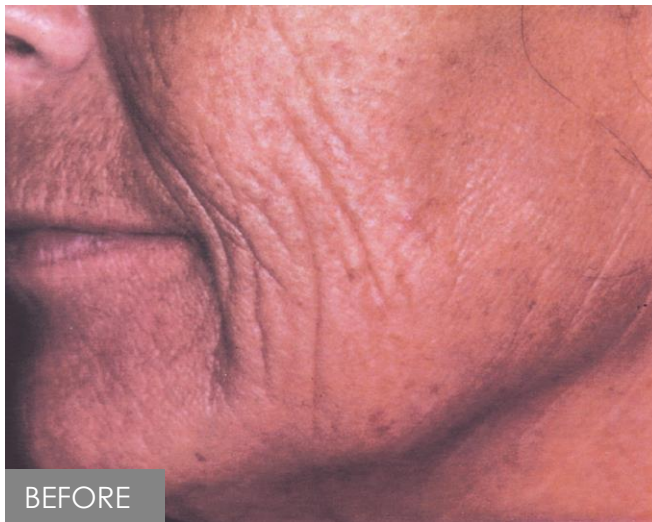


Pulsed Dye Laser

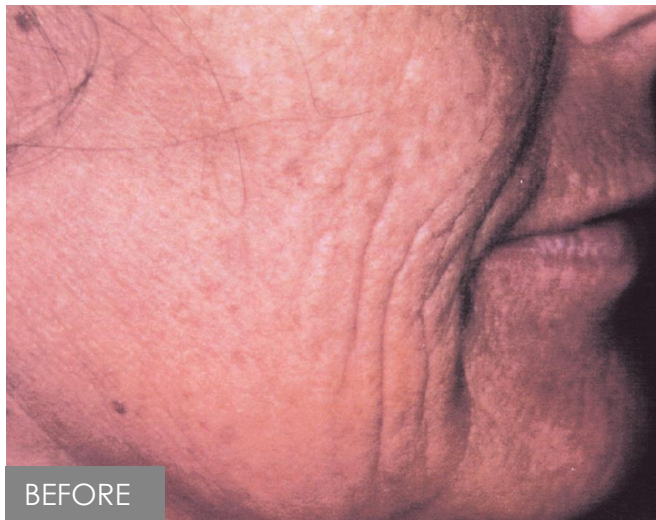

Pulsed Dye Laser

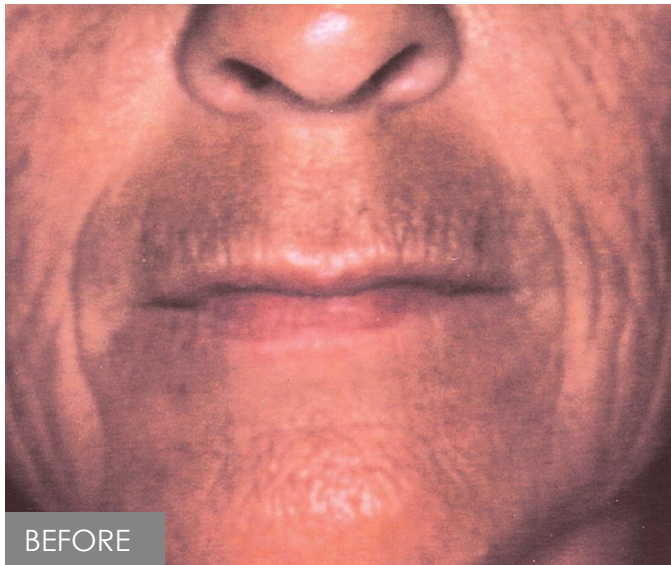

BEFORE

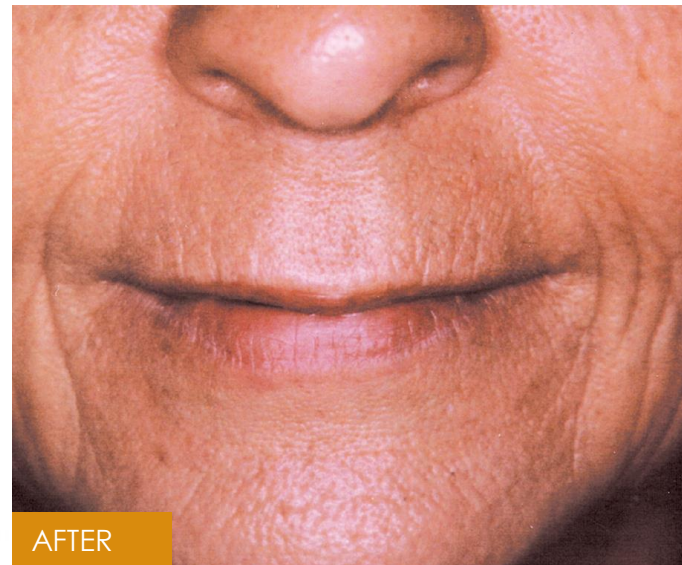

AFTER

Combined with Botox[®] Tx.

Pulsed Dye Laser




PDL alone

Pulsed Dye Laser




Post 1-2 Tx, 7mm spot size, .5 msec 10 J/cm²

Pulsed Dye Laser

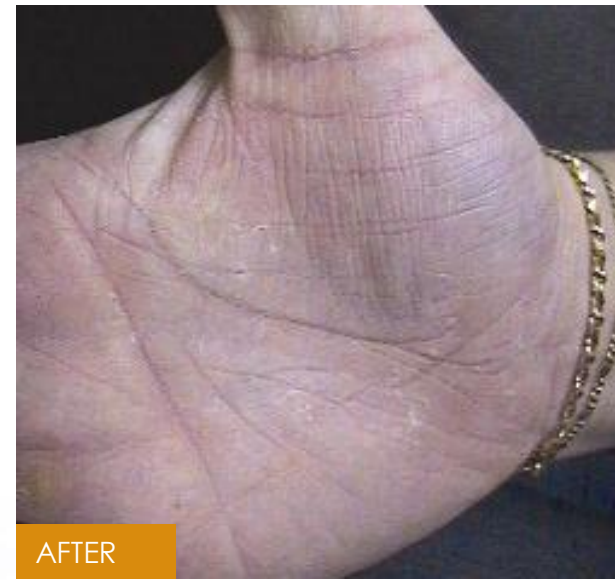

PDL 6-8 J/cm² , .5 msec, 2-4 Tx @ 4-6 week intervals

Pulsed Dye Laser

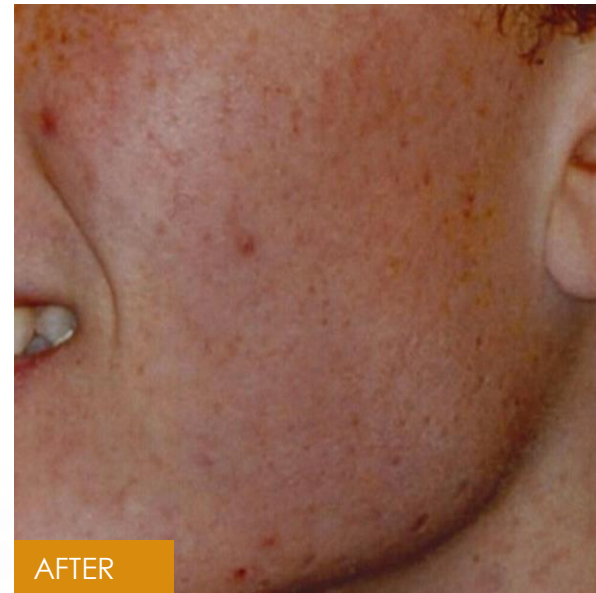

Pulsed Dye Laser



Post Tx, .5msec, 6.5 J/cm²

Pulsed Dye Laser



PDL alone



Post 2 Tx, .5msec, 6 J/cm²

Amico Yasna Pars (Pr.J.S.Co)

CYNOSURE[®]
Cynergy[™]

Nd:YAG



Photos courtesy of E. Tanghetti, MD.

Nd:YAG

